

all-in-ONE 4 Enzyme Detergent Surgical Instrument Cleaner with Conditioners

Highly concentrated, fast, and cost effective pre wash and cleaner.
non-irritating, 100% biodegradable, Phosphate free, and Neutral pH

The all-in-ONE is Neutral pH, non-irritating to eyes and skin, and is 100% biodegradable. Imagine: replacing multiple products, lowering costs, improving the usable life and performance of surgical instruments, while you improve turnaround. The all-in-ONE breaks down and removes all forms of proteinaceous bioburden, cleans and conditions the surface, enhances the passive layer protection of surgical stainless steel, lubricates moving parts, and then rinses clean. 4 enzyme detergents (Lipase, Amylase, Carbohydrase, and Protease) are necessary to break down all forms of bioburden. Enzyme Detergent Surgical Instrument Cleaners that do not deliver a high concentration these four enzymes cannot remove all forms of proteinaceous bioburden. One gallon of the highly concentrated all-in-ONE (3.7 Liters) delivers 512 gallons (1938 liters) of Enzyme Detergent Cleaning and Conditioning power. The dilution rate is .25 to .5 ounces per gallon (1.9 ml liters to 3.8 ml per liter) depending on bioburden level, encrustation, and STAINS.

The all-in-ONE easy FOAM-it lays down a thick blanket of long lasting foam to hydrate and remove all forms of proteinaceous debris, hard water deposits, and encrusted STAINS, for fast cleaning action. When water is added to the easy FOAM-it, the 'FOAM' dissipates providing a clear 'free-rinsing' surgical instrument cleaning and conditioning solution.



easy FOAM-it™
Instrument Pre wash
Cleaner and Conditioners

ONE application is all that is needed for thorough Cleaning and Conditioning. Apply it ONCE. It's all you need for removing STAINS and dried on encrustation. The all-in-ONE is designed for: soaking, pre-cleaning, manual cleaning, ultrasonic cleaning, scope cleaning, and automated washing.



all-in-ONE™
Surgical Instrument
Cleaner with Conditioners

The all-in-ONE cleaners will: break down bioburden, clean the surface, remove STAINS, and lubricate moving parts. The highly concentrated all-in-ONE cleaners are aggressive against your most difficult cleaning challenge and guaranteed to be safe for all types of metal, rigid and flexible scopes, lenses, microsurgical instrumentation and reprocessing equipment.

UPS-FS-32	easy FOAM-it 32 ounce FOAM applicators: 12 per case
Ergo UPS1-1	all-in-ONE 4 gallons with 1 pump per case
Ergo UPS1-5	all-in-ONE: 1 ea. 5 gallon jug
Ergo UPS1-15	all-in-ONE: 1 ea. 15 gallon drum
Ergo UPS1-30	all-in-ONE: 1 ea. 30 gallon drum
Ergo UPS1-55	all-in-ONE: 1 ea. 55 gallon drum

Dilution Rate is .25 ounce to .5 ounce per gallons depending on the level of bio-burden and STAINS.



Customer Service: Joan Hinchman, 1-800-341-7048 Fax: 800-704-2815
email: CustomerService@ergo-Logistics.com

Technical Support: John Temple, 509-747-5027 email: Support@ergo-Logistics.com
Websites: www.ergo-Logistics.com, surgical-instrument-care.com, www.yourCEBA.com